

## Chemetron Wall Type Gas Outlet

### **PURPOSE**

Medical gas outlet serves as terminal supply unit in conjunction with gas piping system, which extends to wall to each department in the hospital. With connecting to regulating devices, it is to deliver intended gas for medical use.

### **SPECIFICATION**

1. Chemetron Console type, Oxygen. (Green)
2. Chemetron Console type, Med Air. (Yellow)
3. Chemetron Console type, Vac. (White)
4. Chemetron Console type, N<sub>2</sub>O. (Blue)

### **WARNING & CAUTION FOR INSTALLATION**


1. Only qualified installing personnel should implement installation.
2. Do not use if there is any damage on the product.
3. Do not use oils, greases, lubricants or any combustible materials on or near this product.
4. Do not smoke in areas where oxygen is enriched.
5. Do not use near any type of flame or flammable/explosive substances, vapors and atmosphere.

### **INSTALLATION INSTRUCTION**

1. Ensure correct gas source is used before installation.
2. Do not use if there is any damage on the product.
3. Ensure all necessary components are available before installation.
4. Ensure serial number is clearly marked on product before installation.
5. Installing Procedure:
  - 5.1 Mount in Rough-In assembly to column
  - 5.2 Connect gas pipeline with correct gas source
  - 5.3 Mount in Finishing assembly in Rough-In assembly
6. Implement leakage test after installation.
7. Insert compatible adapter to ensure gas availability.
8. Visual inspection for proper installation.
9. Complete PRODUCT POST SALE QUESTIONNAIRE and send back to manufacturer.

### **WARNING & CAUTION FOR OPERATION**

1. Only authorized and trained medical personnel should operate this product.

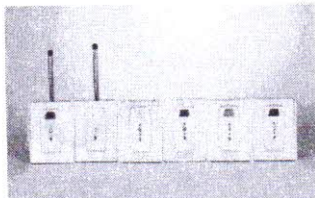

2. Do not use oils, greases, lubricants or any combustible materials on or near this product.
3. Do not use near any type of flame or flammable/explosive substances, vapors and atmosphere.

### **OPERATION INSTRUCTION**

1. Ensure adapter's compatibility for matching this medical gas outlet.
2. Insert adapter to outlet until the lugs of the adapter seat into detent of the outlet keying disc. A positive locking "click" signal indicates complete engagement of the adapter. Press down the color coded area on the outlet for release.

### **MAINTENANCE**

1. Only qualified personnel should repair this product. Inappropriate repairing may damage the product.
2. Use wet cloth with soft detergent for surface cleaning
3. Conduct leakage test every two years after its first use. Replace components when necessary.

### **WARRANTY**

One year warranty.

### **MANUFACTURER**

Pacific Hospital Supply Co., Ltd (PAHSCO)

Head office: 4F, No. 160, Daye Rd. Beitou, Taipei, Taiwan

Manufacturing site: No. 8, Tongke 2nd Rd., Tongliu Township, Miaoli 366, Taiwan

Contact Number: +886 2 2895 5050

Email: [pahsco@pahsco.com.tw](mailto:pahsco@pahsco.com.tw)